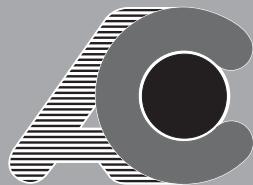
SEPARATORE PER AMALGAMA

**AMALGAM SEPARATOR
SEPARATEUR D'AMALGAME
AMALGAMSEPARATOR
SEPARADOR POR AMALGAMA**

- MANUALE ISTRUZIONI
- OPERATOR'S HANDBOOK
- MANUEL D'INSTRUCTIONS
- GEBRAUCHSANWEISUNGEN
- MANUAL DE INSTRUCCIONES

CATTANI S.p.A.

SEPARATORE PER AMALGAMA

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AMALGAM SEPARATOR

Installation and use

Biological danger, danger of infections from epidemic diseases.

Compulsory direction of flow and rotation.

Inlets i4 and i5 are to be connected to the canister draining pipe and to the spittoon draining pipe respectively. Outlet o4 must be connected to the waste line (figure 1).

Light particles floating on the drained liquid cannot be trapped but heavier ones, among which amalgam waste, are drawn to the bottom of the separator (figure 1).

Aspirated liquids are contaminating for epidemics, therefore, prior to working on used appliances, aspirate a disinfecting agent and let it work for some time; wear disposable gloves, mask, goggles and overall in order to protect yourself against accidental splashing.

The amalgam separator is to be emptied as follows:

tilt the separator very slowly (to avoid dispersing the amalgam) to drain most of the liquid (figure 2), take off the cover item 201060 by rotating it in

the direction shown by arrow A (figure 3); put the disposable amalgam container item 043010 on top of the separator item 201100 and keep it pressed (figure 4); quickly turn upside down and shake both containers so that all amalgam waste drops into the disposable container (figure 5).

Fill the disposable container with a disinfectant.

Cover 201020 is safe, however we suggest you to fasten the cover with tape (figure 6) **always keep the container in an upright position and handle it with care.**

Disposal of recovered amalgam (harmful, toxic and contaminating substance) must be carried out by authorised companies. A list of authorised companies is available at the Chambers of Commerce. For further information, address to local authorities and to professional associations.

The authorised retailer and the manufacturer are at your disposal for further information, advice and spare parts. All maintenance trials should be carried out on brand new appliances, before any use, in order to avoid contamination.

This appliance does not comply with international regulations; on the contrary, our models **ISO Hydrocyclones** comply with **UNI EN ISO 11143** regulations.

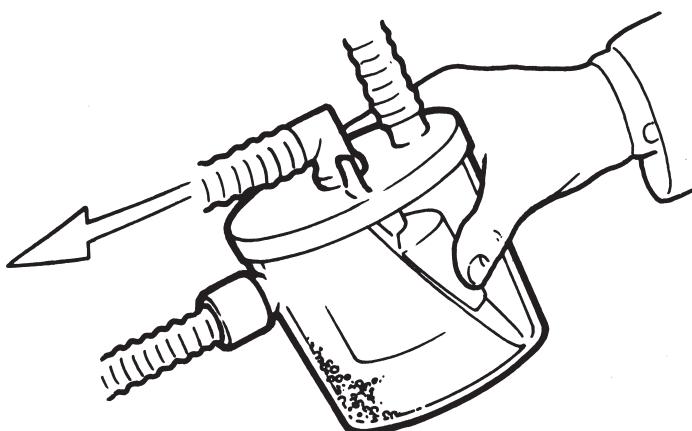
Fig. 2

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